




Joerns
Oxford®

User Instruction Manual

Oxford® Quickfit Deluxe Disposable Sling

To avoid injury, read user manual prior to use. For alternative languages, contact your authorised service provider.

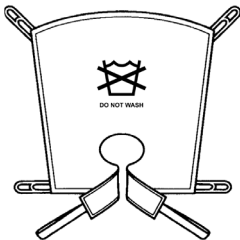

Introduction

The Oxford® Quickfit Deluxe Disposable sling is a development of the standard Quickfit design and is suitable for the vast majority of patients. Contoured leg sections provide increased hip and thigh support and it can be used for amputees following careful assessment. In the interests of avoiding cross-infection, this sling is specifically designed for single patient use only.

NOTE: DO NOT WASH DISPOSABLE SLINGS

Statement of Intended Use

A sling is an item of moving and handling equipment that is used with a mechanical lift in order to facilitate the transfer of a patient. It comprises a specially designed and constructed piece of fabric that is placed under and/or around a patient before being attached to the spreader bar/cradle of a lift to raise, transfer and lower the patient. When selected and used correctly, a sling and lift combination will achieve a safer transfer and reduce the risks associated with manual handling.

It is the responsibility of a competent person to conduct a thorough risk assessment prior to using any sling, to ensure that the sling choice, method of positioning in the sling and procedure for transfer has been correctly determined for the patient. For further guidance, please contact your authorised Oxford Service Provider or Joerns Healthcare.

FOR USE WITH PASSIVE LIFTS ONLY.


Serious Incident Reporting

In the event of a serious incident taking place during use of this product, affecting the patient and/or caregiver, it must be reported to the product manufacturer or authorised distributor. Should the incident take place within the European Union (EU), it must also be reported to the local competent authority within the member state.


⚠ WARNING

- **OXFORD RECOMMENDS THE USE OF GENUINE OXFORD PARTS. Oxford sling and lift products are designed to be compatible with one another. For country specific guidance on sling use and compatibility, please refer to the sling label or contact your local market distributor or Joerns Healthcare.**
- **For the safety of the patient and carer; before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.**
- **CHECK sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.**
- **DO NOT alter slings. Destroy and discard worn slings.**
- **NEVER leave a patient unattended.**
- **DO NOT exceed the rated capacity of the sling or lift.**
- **DO NOT attempt to re-position a patient by pulling on the sling loops.**


Fitting the Sling from a Seated Position

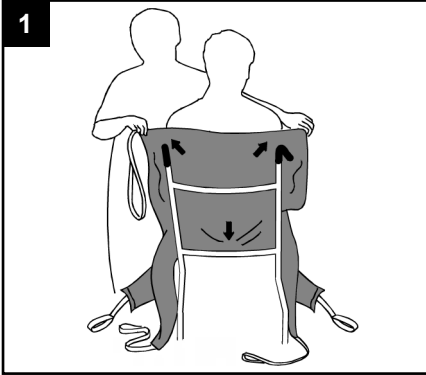


Make sure the hand grips and seams are on the outside of the sling. Feed the sling down the back of the patient, leaving the commode aperture at the base of the spine. Check the sling is square across the shoulders.

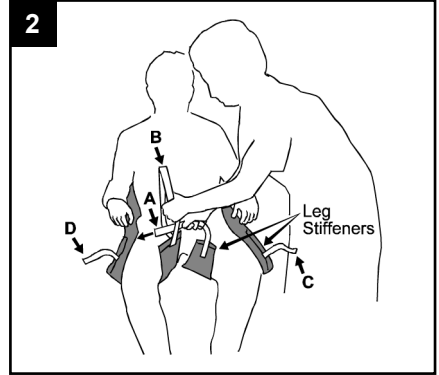

Bring the longest leg piece stiffener forward so that it is parallel with the long bone of the thigh. Ensuring that the leg piece is not twisted, take the shortest leg piece stiffener under and up between the legs. Repeat this procedure for the other leg. Cross strap A through strap B.

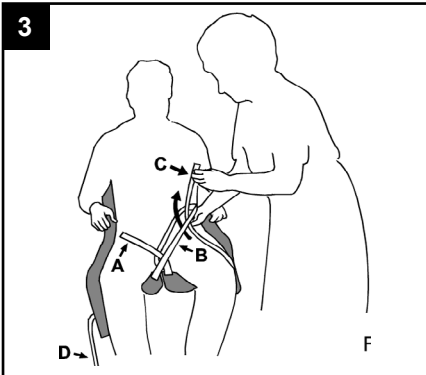


Pass strap C up through strap B then repeat for other leg using strap D through strap A.

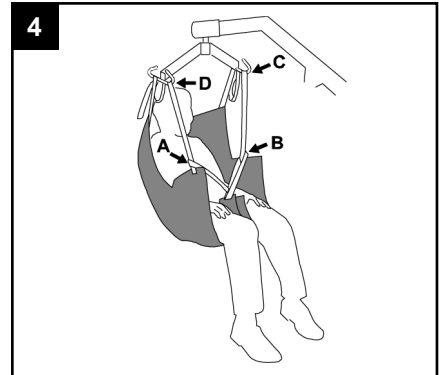


Check the sling is smooth under the patient and the position is comfortable. Move the hoist into position and attach the leg straps to the spreader bar using the coloured loops. This maintains the patient in an upright sitting position. The lift may move towards the patient as you do this. When reseatting the patient the hand grips may be used to ensure a good posture.

Fitting the Sling from a Lying Position

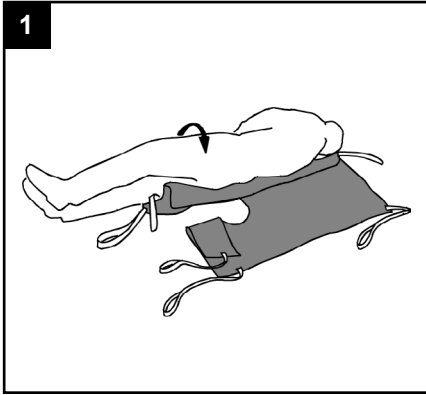

Safely position the client onto the sling, ensuring that the commode aperture is at the base of the spine.

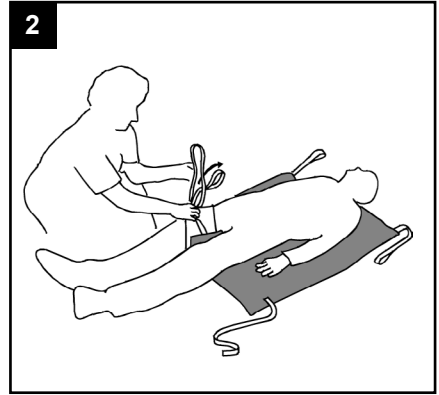

Cross leg strap A and B under and between the legs and feed A through B. Feed C and D through the long loops on A and B. Then proceed to attach the sling to the spreader bar in the same way as the sitting position.

Technical Specifications

STANDARD	BS EN ISO 10535
----------	-----------------



This product is a Medical Device in accordance with EU Medical Device Regulation 2017/745.

Sizing & Safe Working Load

SIZE	SWL
SMALL (RED)	500lbs / 227kg
MEDIUM (YELLOW)	500lbs / 227kg
LARGE (GREEN)	500lbs / 227kg

For more information on sling dimensions, please contact us. For guidance on sling application, visit www.oxfordslingsselector.co.uk.

Washing Instructions & Safety Checks



DO NOT wash



DO NOT dry clean



DO NOT tumble dry



No smoking



DO NOT iron



⚠ WARNING

Disposable slings are patient specific and are designed not to be washed. Washing will compromise the integrity of the sling and therefore make it unsafe for use. Scan QR code or [click here](#) for important sling safety checks and information.

PLEASE NOTE: Additional slings are available to meet individual patient needs. We advise that you always seek the advice of a trained clinician, authorised Oxford distributor or Joerns Healthcare prior to purchase or use.

End of Life Disposal

Slings and associated material accessories should be sorted as combustible waste in accordance with local or national regulations.

THIS PAGE IS INTENTIONALLY BLANK

THIS PAGE IS INTENTIONALLY BLANK

