



Parts Manual




Oxford® Mermaid

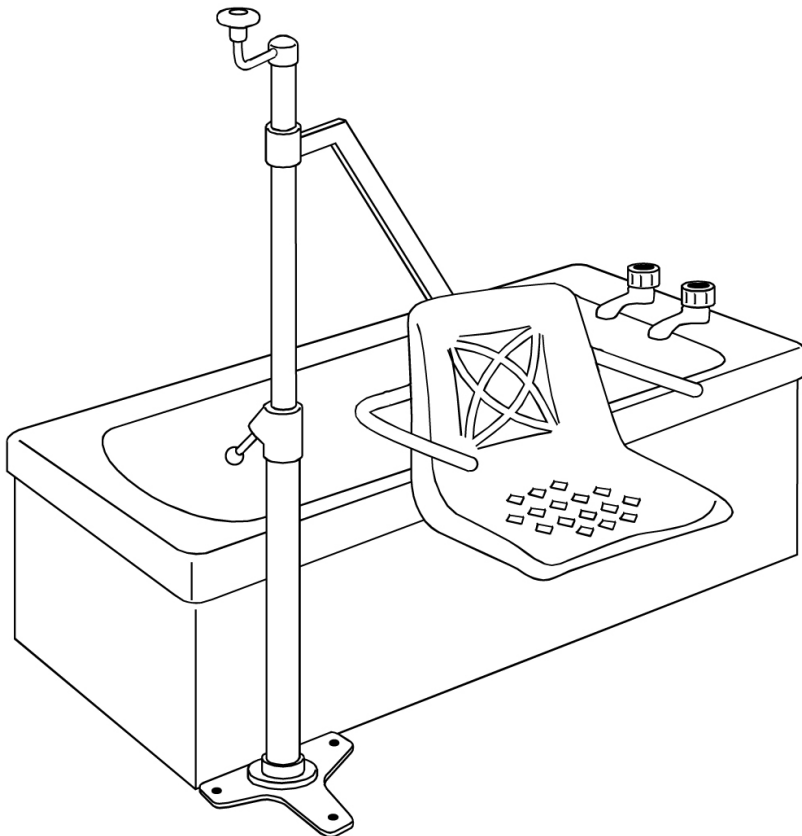



NO	PART NUMBER	DESCRIPTION	QTY	SERIAL NO.
1	324-02010	Seat arm security screw (M8) (1997 onwards)	1	
2	808003.10052	Seat support clamp	Kit	
3	381-40020	Thrust washer - PTFE	1	
4	808003.10056K	Locking handle complete	Kit	
5	80067	Locking mechanism cover	1	
6	808003.10063	Column/baseplate fixings	Kit	
7	808003.10076	Attendant winding handle complete	Kit	
8	648001.10351	Sub assembly square tube	1	
9	219-19015K	Circlip	10	
N/S	0P10002	Electric column assembly complete (Without charger) (not shown)	1	
N/S	0P10000	Manual column assembly complete (not shown)	1	
N/S	39001298	Electric column assembly with UK charger (not shown)	1	
N/S	39001299	Electric column assembly with EU charger (not shown)	1	

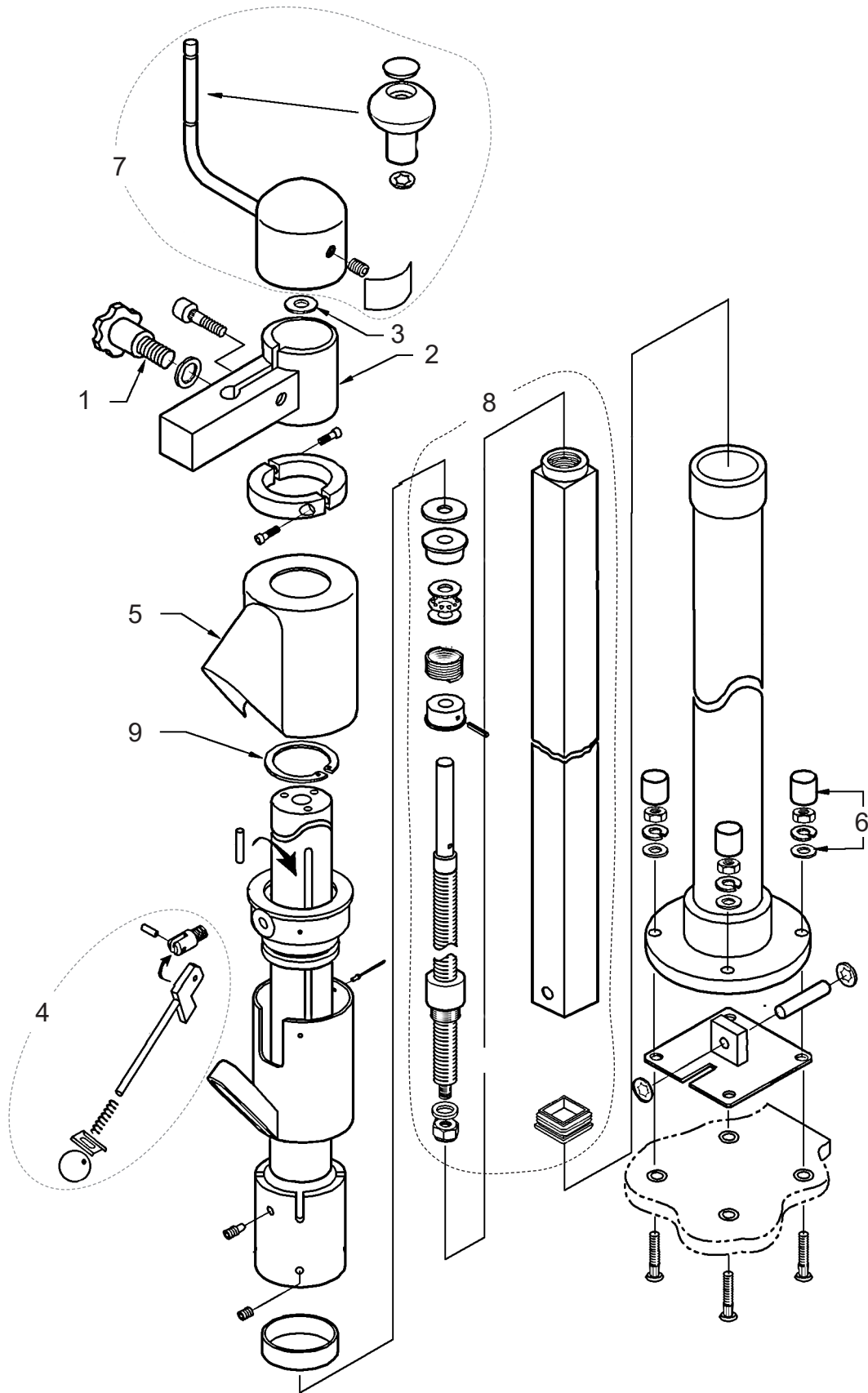


NO	PART NUMBER	DESCRIPTION	QTY	SERIAL NO.
1	OP10012	75mm column extension (complete with fixings)	Kit	
2	808003.10064	Universal baseplate for concrete floor	Kit	
3	808003.10065	Universal baseplate for wood floor	Kit	

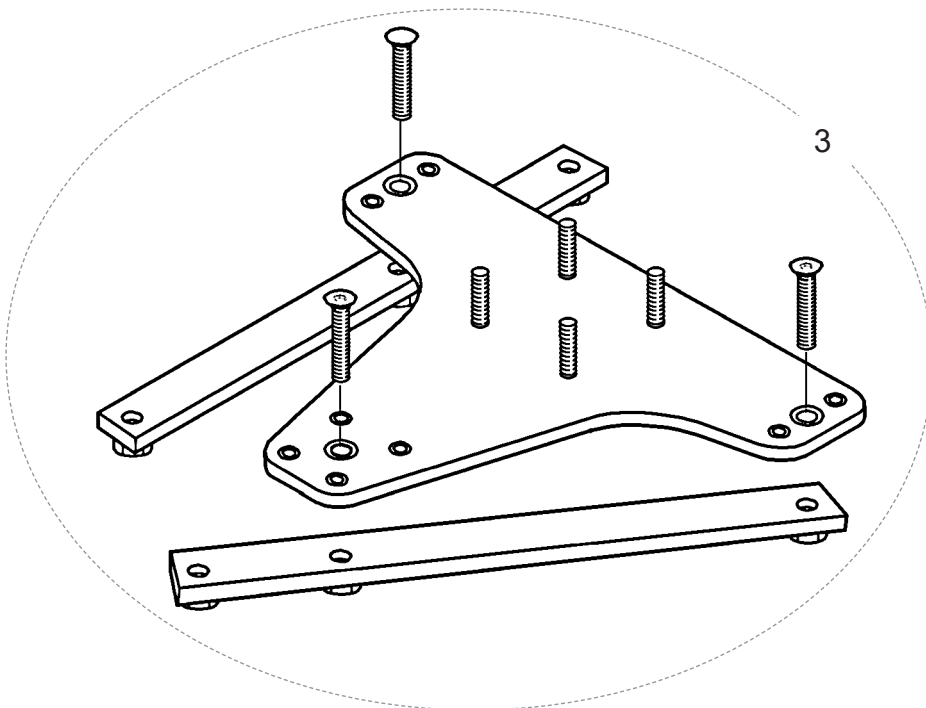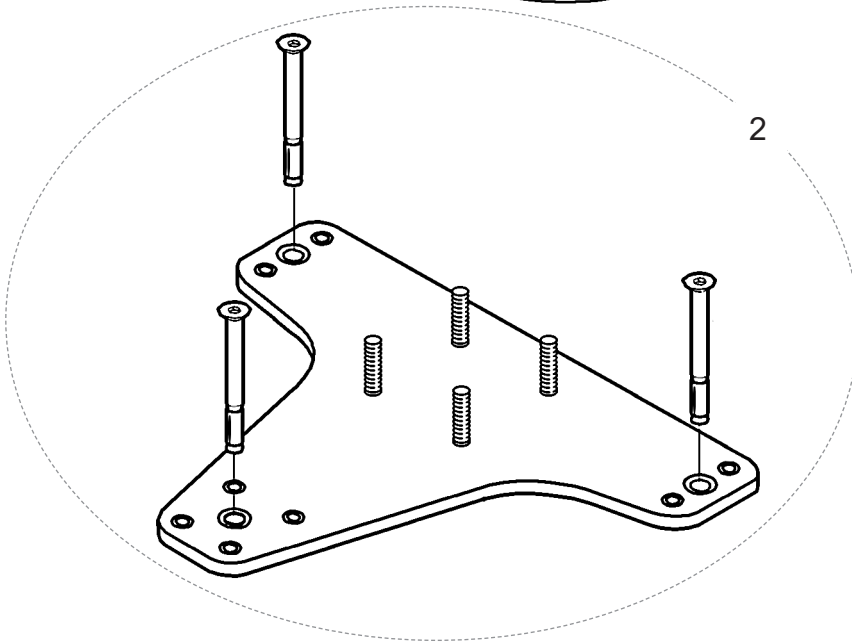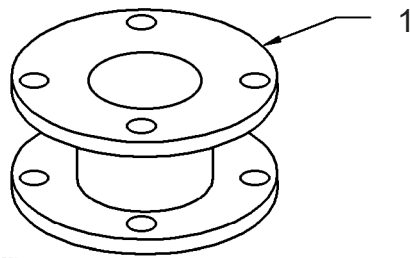

NO	PART NUMBER	DESCRIPTION	QTY	SERIAL NO.
1	324-02010	Seat arm security screw (M8) (1997 onwards)	1	
2	808003.10070	Mermaid plastic seat (Standard)	Kit	
3	808003.10071K	Safety arms and pivot complete (Standard and Commode)	Kit	
4	323-11010	Handgrip	1	
5	808003.10074	Ranger/Commode plastic seat complete (seat, backrest & fixings)	Kit	
6	808003.10077	Mermaid seat fixings & trims (Standard)	Kit	
7	808003.10078	Commode seat fixings & trims	Kit	
8	OP10019	Mermaid Commode Seat Kit – Side Fit	Kit	
N/S	OP10020	Mermaid Commode Seat Kit – End Fit	Kit	
9	OP10004	Mermaid Standard Seat Kit – End Fit	Kit	
N/S	OP10003	Mermaid Standard Seat Kit – Side Fit	Kit	
N/S	750210J2	Lap strap (not shown)	1	

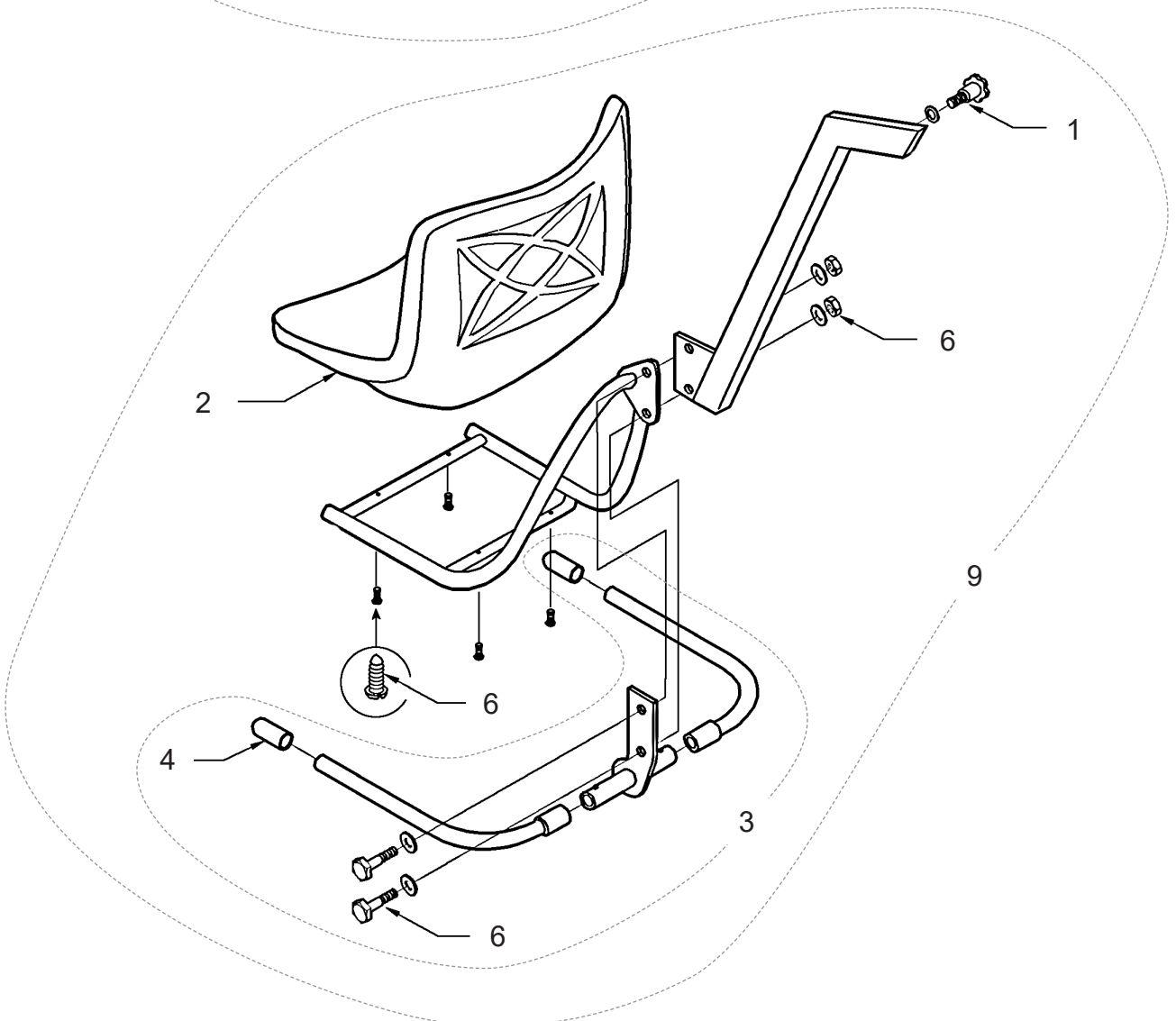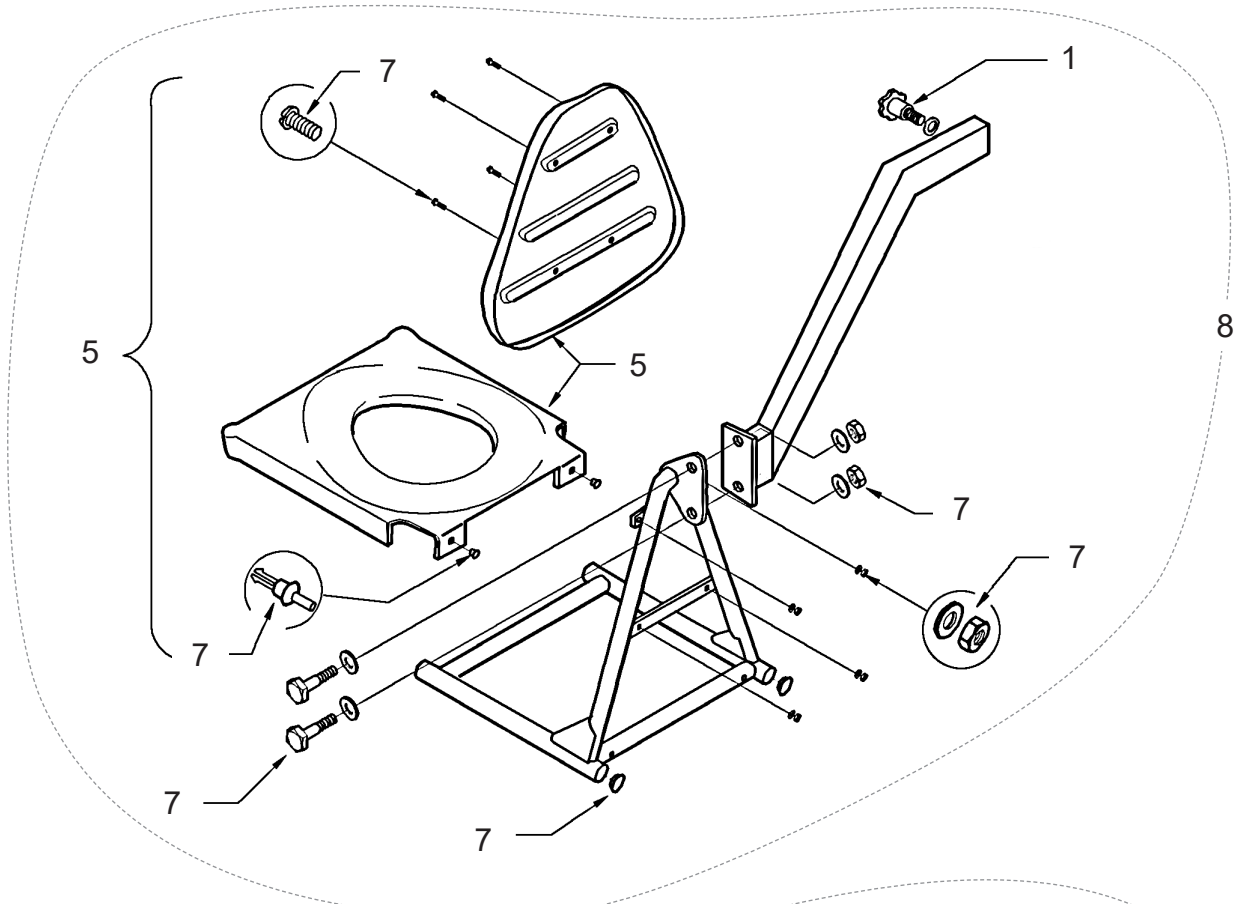

NO	PART NUMBER	DESCRIPTION	QTY	SERIAL NO.
1	MX/3674	Plastic main cover	1	
2	EL/3868	Hand control socket - (3 pin axle - wired)	1	
3	EL/3872	Emergency raise switch - wired (new switch - oblong)	1	
4	EL/3866	Power/charger plug (straight) - wired	1	
5	808003.10088	Motor complete - including Woodruff key	Kit	
6	EL/3870	Microswitch limit switch - wired	1	
7	200001.10050	Control PCB - Clarkes with edge connector	1	
8	MX/3607K	Motor to lead screw coupling kit	1	
9	MX/3885	Hand control	1	
10	TS-2923W	Power pack complete	1	
N/S	OP10005	Retro-fit kit (excluding charger) (not shown)	Kit	
N/S	338-00090	Charger (230V) (not shown) UK	1	
N/S	EL/3716	Power/charger socket - wired (not shown)	1	

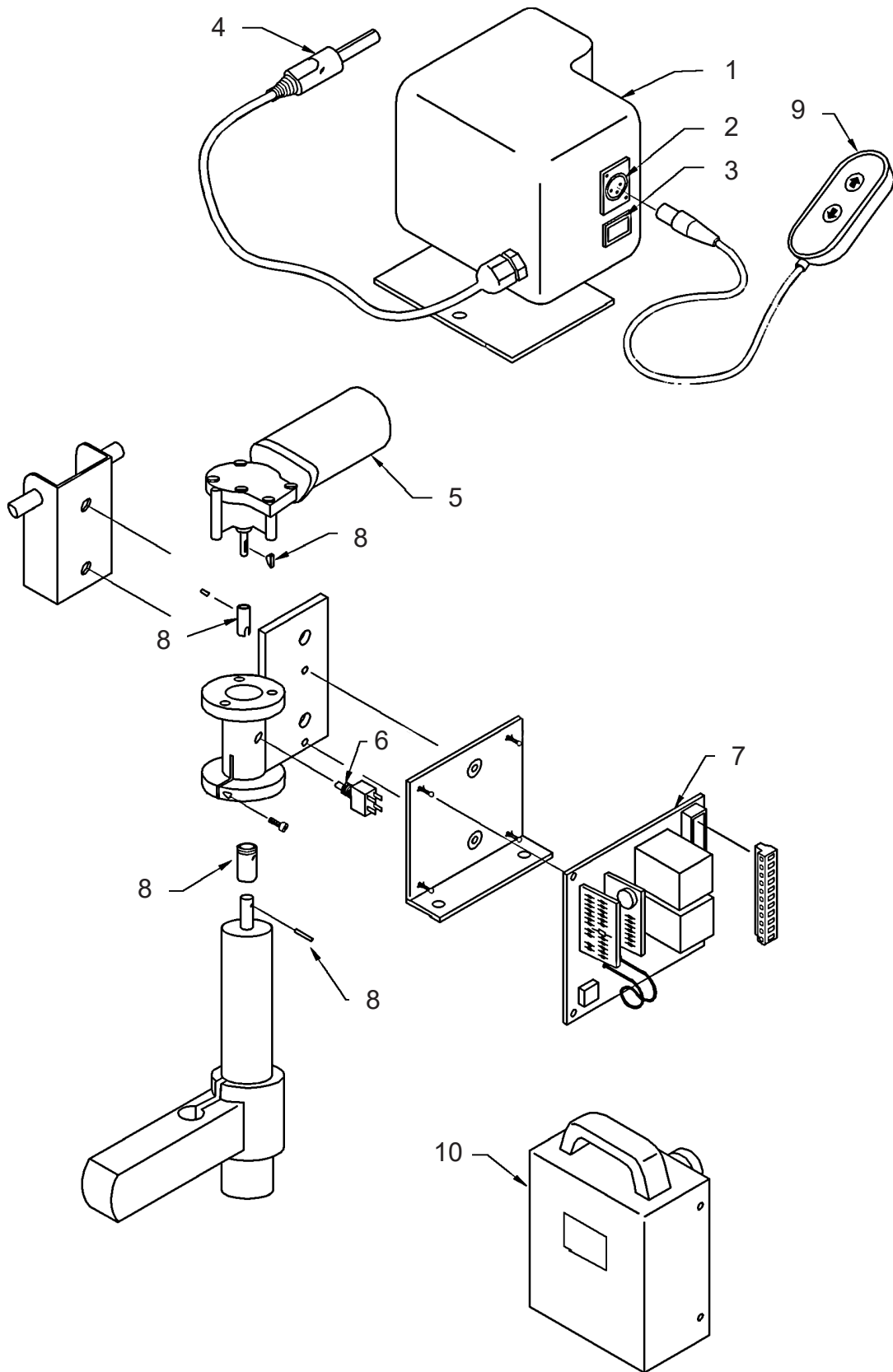

NO	PART NUMBER	DESCRIPTION	QTY	SERIAL NO.
1	808003.10079	Ranger seat support arm - side fit (Mermaid only) (Includes arm safety latch)	Kit	
2	808003.10080	Ranger seat support arm - end fit (Mermaid only) (Includes arm safety latch)	Kit	
3	323-11010	White hand grip	1	
4	808003.10074	Seat, backrest & fixings (Ranger and Commode)	1	
5	808003.10157	Securing hook complete	Kit	
6	311-41110	Castor 100mm braked (without fixings)	1	
7	311-42110	Castor 100mm plain (without fixings)	1	
8	OP10007	Ranger Commode & Transporter - End Fit	1	
	OP10006	Ranger Commode & Transporter – Side Fit	1	
N/S	750210J2	Lap strap (not shown)	1	

