User Instruction Manual Oxford® Low Profile Weigh Scale

To avoid injury, read user manual prior to use. For alternative languages, contact your authorised service provider. Always refer to Joerns.co.uk for latest revision.

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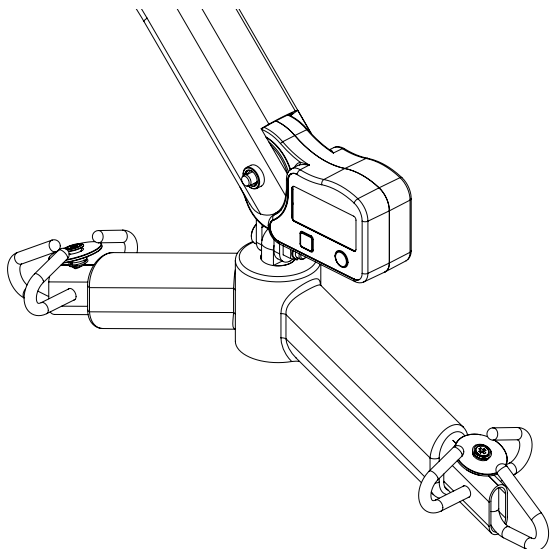
Contents

| | |
|---|----|
| 1. Introduction | 3 |
| 2. Component Parts | 4 |
| 3. Assembly | 5 |
| 4. Operation | 7 |
| 5. Operation Features | 8 |
| 6. Cleaning | 8 |
| 7. Technical Specifications | 9 |
| 8. Battery Replacement | 11 |
| 9. Maintenance Inspection and Test..... | 12 |
| 10. Warranty | 12 |
| 11. Logbook for Scale..... | 13 |

1. Introduction

About your scale:

The Oxford Digital Scale is a technologically advanced, microprocessor controlled, digital weighing device. It features two large, easy to read, Liquid Crystal Displays (LCD) that show the patient's weight from either side.

The controls are simple and straightforward to use. The Oxford Digital Scale is powered by an inexpensive, common 9 volt battery.

The Oxford Digital Scale can be used in conjunction with the Oxford Presence and Stature. The model numbers for the scale are:

| | |
|-------------------|--|
| OXF-UNISCALE100 | 4-point cradle with scale (calibrated for UK) |
| OXF-UNISCALE100EU | 4-point cradle with scale (calibrated for non UK) |
| OXF-UNISCALE101 | 6-point spreader bar with scale (calibrated for UK) |
| OXF-UNISCALE101EU | 6-point spreader bar with scale (calibrated for non UK) |
| OXF-UNISCALE102 | 4-point Adaptive power cradle with scale (calibrated for UK) |
| OXF-UNISCALE102EU | 4-point Adaptive power cradle with scale (calibrated for non UK) |

The CE mark (Europe Only):

The Oxford Digital Scale carries the CE mark and complies with the following EC Directives:

- EMC Directive (2014/30/EU)
- NAWI Directive (2014/31/EU)
- RoHS Directive (2011/65/EU)

Intended Use:

The Oxford Low Profile Weigh Scale is a RoHS compliant precision scale for use in portable patient weighing systems. The scale is designed to be connected between patient hoist mechanism and the patient sling apparatus. The Oxford Low Profile Weigh Scale is NOT intended to be used during patient transportation.

Serious Incident Reporting

In the event of a serious incident taking place during use of this product, affecting the patient and/or care-giver, it must be reported to the product manufacturer or authorised distributor. Should the incident take place within the European Union (EU), it must also be reported to the local competent authority within the member state.

3. Assembly

Your weigh scale comes fully integrated with the 6 point (loop) spreader bar, 4-point (non-powered) cradle or a 4-point adaptive power cradle (powered) cradle.

Attaching the scale/cradle assembly to the lift is quick and easy and will require a small blade screwdriver.

You need to first remove your existing spreader bar or cradle from the lift.

Removal of the spreader bar/cradle:

To attach your new Oxford digital scale you may first need to remove the spreader bar/cradle from the lift by removing the quick release pin that retains the 4-point cradle or 6-point spreader bar (Fig. 2). A small flat blade screw driver is the only tool required.

Caution: Prior to removing the quick release pin, the cradle needs to be supported by a stationary object such as a work bench or table. This will prevent the cradle from dropping suddenly preventing an injury or damage to the cradle.

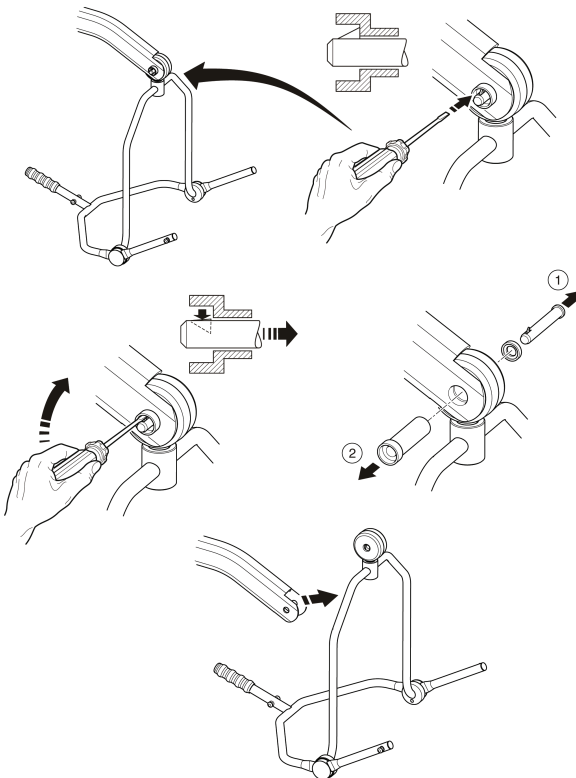
Fig. 2

Connection of the Spreader Bar/Cradle:

To connect, raise the spreader bar/cradle up towards the boom slot. Once aligned in the boom slot reinsert the quick release pin fully into the fulcrum bush until it locks into position. **YOU MUST CHECK THAT THE PIN IS SECURELY FITTED;** a noticeable “click” will sound when the connection has been made secure.

To do this, press your finger into the barbed end. If the pin remains stationary the lift is safe to use (Fig. 3).

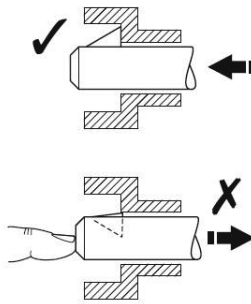
Fig. 3

⚠ WARNING

The pin and fulcrum bush must engage through the boom slot. If the fulcrum bush becomes misaligned this may result in serious injury/death.

4. Operation

The Oxford scale has been designed and certified to weigh up to 400kg in 0.2 increments. It has been certified to weigh in temperatures between 5°C/35°C.

To weigh a patient follow the instructions below:

1. To power up the scale press and release the ON/ZERO button. The display will show either 0.0 or a random measurement.
2. Attach the sling to the spreader bar/cradle that will be used to weigh the patient (this is necessary so the weight of the sling is not included with the patient's weight).
3. Press and hold the ON/ZERO button for approximately 2 seconds until "ZERO" is displayed. The scale will zero and is now ready to weigh a patient.

NOTE: The upper left corner of the display will have the ">0<" symbol. This indicates that the scale has been properly zeroed. No more than 40kg can be tared to zero.

4. Remove the sling from the spreader bar/cradle and place the sling around/under the patient for proper lifting.
5. Reattach the sling to the spreader bar/cradle; raise the patient so that they are suspended from the lift.

NOTE: The patient must not be touching anything such as a chair or bed since this will cause an error in weighing.

6. Once the digital display has stabilized, record the weight and then lower the patient onto the appropriate surface (i.e. bed, chair).

NOTE: The upper left corner of the display will have the ">□<" symbol. This indicates that the weight being measured is stable.

7. The scale has been programmed to turn off after 1 minute to conserve battery life. The zero (TARE) will be maintained. To restart the scale prior to lowering the patient, push the ON/ZERO pad momentarily (do not hold for >2 seconds or the scale will be zero under the weight of the patient). The display will show the start up screen and then show the patient's weight

NOTE: To restart the scale, be sure to push the ON/ZERO. If the scale is "turned on" and the pad is pushed again or pressed for more than 2 seconds, the scale may go to zero (0.0) and the patient's weight will be lost.

8. The RECALL feature is designed to retain in the memory the last stable weight for a patient

NOTE: Pressing the RECALL button will display the last viewed stable weight, indicated by the MEM symbol in the upper left corner of the display.

5. Operation Features

- All functions are accessible to the user without opening the case.
- Starting or zeroing the scale is done through a single touchpad located on the front of the case.
- Stable readings are achieved through digital averaging. Once the digital display has stabilised, the stable weight will be held. To make the scale active again, push the ON/ZERO pad momentarily. The held weight is stored in a non-volatile memory for later recall if required.
- Display turns off after 60 seconds of inactivity to conserve battery life.
- Recall of the last held weight is available by pressing the RECALL pad. This allows a reading of the last patient's weight, after the patient has been removed from the scale.
- Battery change is made easy with external access to the battery holder.

6. Cleaning

Clean with ordinary soap and water and/or any hard surface disinfectant. Harsh chemical cleaners or abrasives should be avoided as these may damage the surface finish of the product. Avoid wetting any of the electrical parts. After cleaning, the unit should be thoroughly dried.

CAUTION

Do not scratch the display window.

Do not immerse the Weigh Scale in liquid or spray liquid directly on the unit.

Do not use pressurised water or steam on the weigh scale.

End of Life Disposal

Battery: All batteries in the product and any accompanying accessories must be recycled separately. Batteries should be disposed of in accordance with local or national regulations.

Slings: Slings and associated material accessories should be sorted as combustible waste in accordance with local or national regulations.

7. Technical Specifications

- ElectronicsMicroprocessor controlled.
- Power9 volt alkaline battery PP3.
- Battery LifeMinimum 3000 cycles (60 seconds default)
- Display.....LCD (liquid crystal display) 70 x 30mm screen with 25mm high characters
- Weighing UnitsKilograms (kgs).
- Load CellRated at 400 kgs.
- CaseABS

Weights:

- Oxford Low Profile Weigh Scale0.9 kgs
- Oxford Low Profile Weigh Scale fitted to 6 Point Spreader Bar2.0 kgs
- Oxford Low Profile Weigh Scale fitted to 4 Point Cradle6.8 kgs
- Oxford Low Profile Weigh Scale fitted to the APC.....9.3 kgs

WARNING

Do not exceed the capacity of the lift. Use above this capacity is unsafe and may result in serious injury or death.

CAUTION: The digital weigh scale is to be used in a clean, dry, temperature controlled indoor environment.

KEY SYMBOLS:

The following symbols are used on the charger, control unit and battery:

Type B equipment, as per EN 60601-1.

Class 2 equipment.

For indoor use.

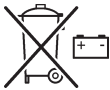
ATTENTION, consult accompanying documents.

Refer to the user instructions before use (Blue background).

WEEE Symbols:

The disposal of batteries should not be mixed with general household waste. Batteries should be disposed of in accordance with local or national regulations.

The disposal of electronics should not be mixed with general household waste

The crossed-out wheeled bin means that within the European Union the product must be taken to separate collection at the product end-of-life. This applies to your device but also to any enhancements marked with this symbol. Do not dispose of these products as unsorted municipal waste.

WEEE Producers registration number WEE/GG0464R2

8. Battery Replacement

The Oxford Electronic Digital Scale is powered by a 9 volt battery. A new battery will provide approximately 3000 readings before replacement is necessary. The low battery 'LOBAT' indicator is displayed when battery replacement is required.

To Replace Battery:

1. Push black pad on the battery compartment (located on the underside of the weigh scale) in the direction of the arrow. Pull out to withdraw the battery compartment.
2. Battery holder will slide out.
3. Remove old battery and replace with a new one.
4. Push battery holder back into scale.

9. Maintenance, Inspection & Test

Joerns Healthcare recommends that a thorough inspection and test of all Oxford lifts and lifting accessories is carried out every six months. Joerns Healthcare recommends maintenance, inspection and certified testing, is only carried out by a competent service provider. Refer to the Lift user manual for a checklist of items to be inspected and tested.

Important Note (European Requirement Only):

The Oxford Weigh scale is a Class III compliant weighing device and conforms to the NAWI Directive 2014/31/EU and has been certified by NMO, certification number UK3114.

Re-calibration is generally not required, but should be verified periodically to ensure accuracy. The recommendation for calibration check is at least once every 12 months, or as individual maintenance policy requires.

Servicing, Repairs, Inspections and Testing:

Joerns Healthcare has an established network of reputable distributors and service technicians who will be pleased to handle all your purchasing, warranty, repair and maintenance inquiries.

10. Warranty

Joerns Healthcare has an established network of authorised distributors and service agents who will be pleased to handle all of your purchasing, warranty, repair and maintenance enquiries. It is recommended that our products are commissioned by your authorised distributor or service agent.

The distributor or service agent operates the warranty programme, so it is important to keep a record of their name, address and telephone number so they can be contacted should any problem arise. If you are in any doubt as to where your lift was purchased, Joerns Healthcare can trace the supplier if you quote the serial number of the lift.

All warranty claims are subject to the correct use and maintenance in accordance with the user instruction manual provided with each lift. Damage caused by use in unsuitable environmental conditions or failure to maintain the product in accordance with user and service instructions is not covered. Your statutory rights remain unaffected.

REMEMBER: Contact your authorised distributor or service agent for purchases, repairs, servicing and certified maintenance.

Your distributor:

11. Logbook for Scale

Non-automatic weighing instruments:

| | |
|---|----------------------------------|
| Verification interval (years) | 1 Year |
| Name and address of customer (end user) | |
| Scale model | OXF-SCALE-LPU / OXF-SCALE-LPU-EU |
| Scale serial number | |
| Classification of the scale | III |

Purpose of the logbook:

- The logbook retains all information relevant to the inspection, repair or modification of the instrument.
- The logbook can be provided by Joerns, the approved inspector or the approved repair engineer.
- The logbook must accompany the scale at the place of use.
- The logbook must always accompany the scale, even if the scale is mobile.

Responsibility of equipment holder (end user):

- The end user must make this logbook available at all times for the local regulatory authority, the approved inspector, or approved engineer.
- The responsibility for producing the logbook lies with the end user of the logbook.
- The end user is responsible for looking after the integrity of the logbook.
- A missing or damaged logbook will result in signature refusal of the periodic verification and/or refusal by the approved engineer to work on the scale.
- If the current logbook is replaced, the approved inspector or engineer must note this event in the new logbook
- It is the end user responsibility that the approved bodies for verification fill in the logbook; the end user should sign the logbook after the signature of the approved inspector or engineer.

| | |
|---|--|
| *Initial verification (check box) | |
| Date verified (DD/MM/YY) <input type="checkbox"/> | |
| Verification body - EU versions only | |
| Organisation ID # Name/address of service provider | |
| Represented by | |
| Signature of service provider | |

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Joerns Healthcare Limited